Excelite™

FREEDOM

Your choice to perfect vision

The Individual Freeform Back Surface
Progressive Addition Lens Series.

FREEDOM CLASSIC
FREEDOM NEO-VISTA

HILUX OPTICAL

FREEDOM Series : Lens concept

Individual Freeform Back Surface PALs

- FREEDOM CLASSIC – Freeform Back Surface PALs
- FREEDOM NEO-VISTA – WFT + more individual concepts

Available : CR-39 – TVX – 1.6/42

G / B : SunSensors – Transitions IV 1.5 – Transitions V 1.6/42

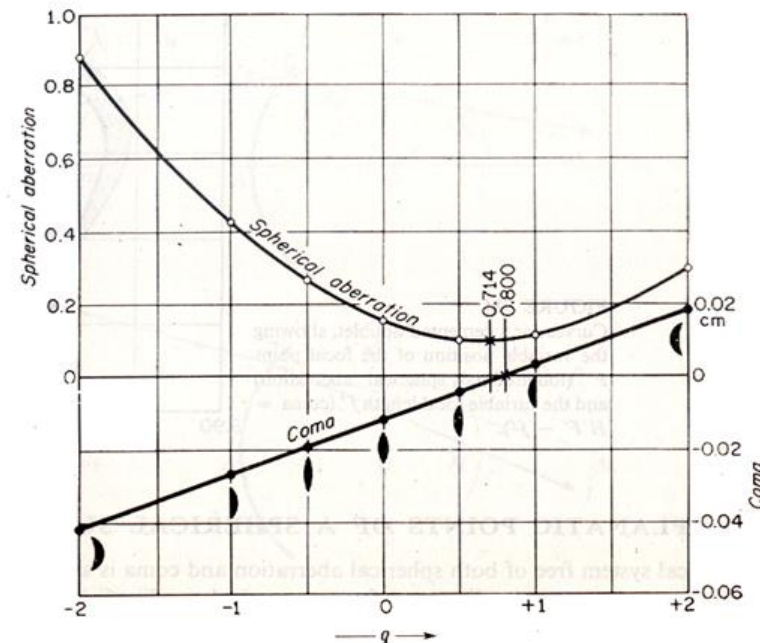
G / B : Polarview 1.5

Power range : SPH +6.00 D to -6.00D CYL -3.00D for 1.5 and TVX

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
Graphs comparing coma with longitudinal spherical aberration for a series of lenses having different shapes.



FREEDOM CLASSIC

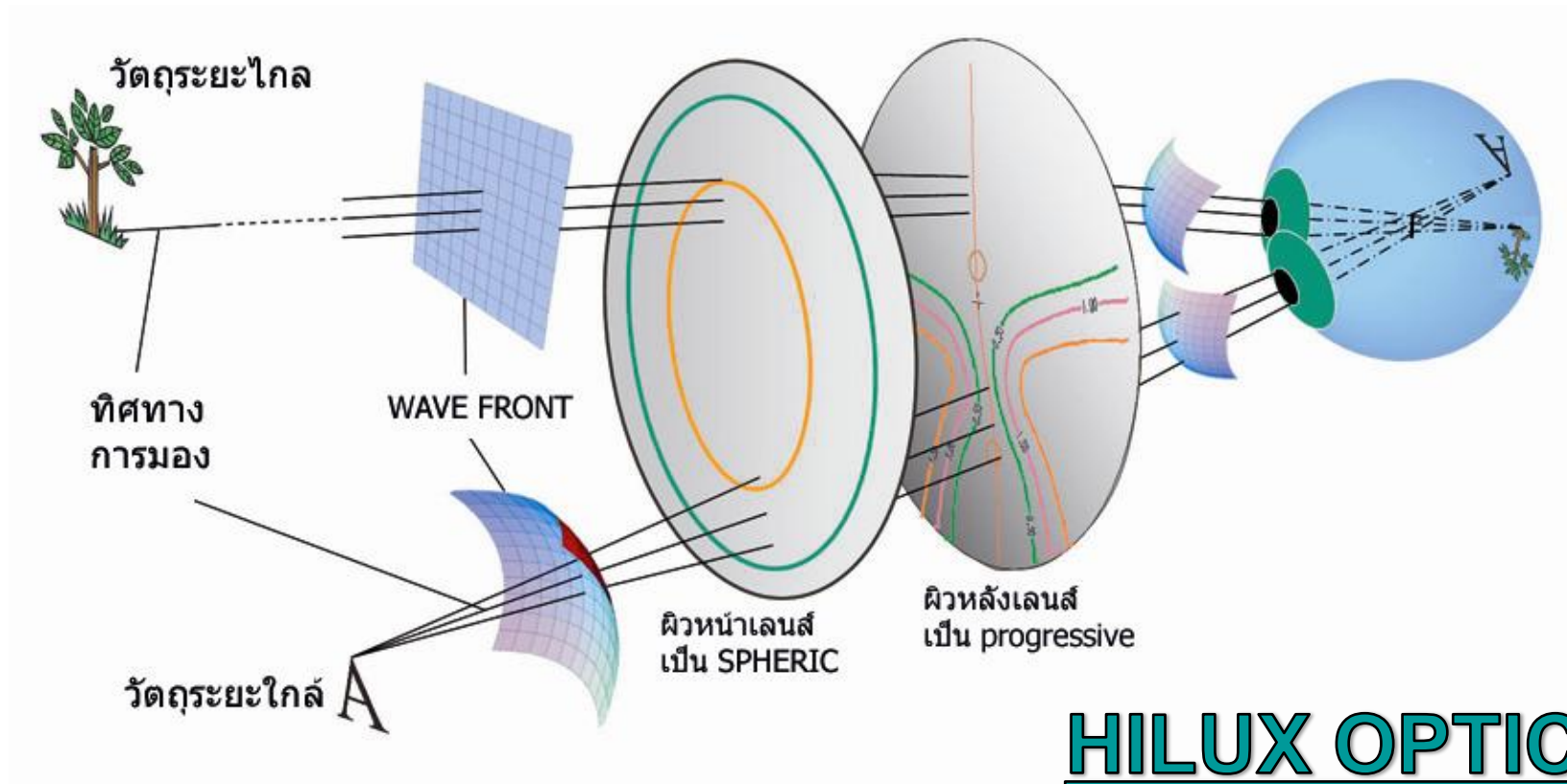- Progressive Back Surface :
Front SPH – PAL back surface

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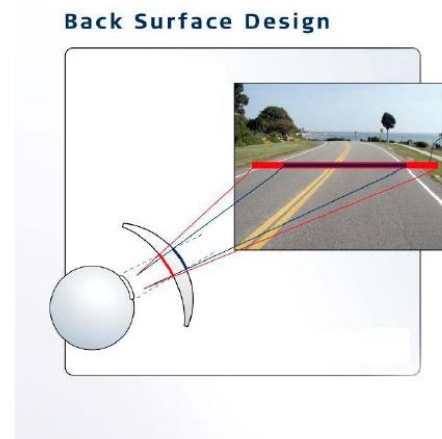
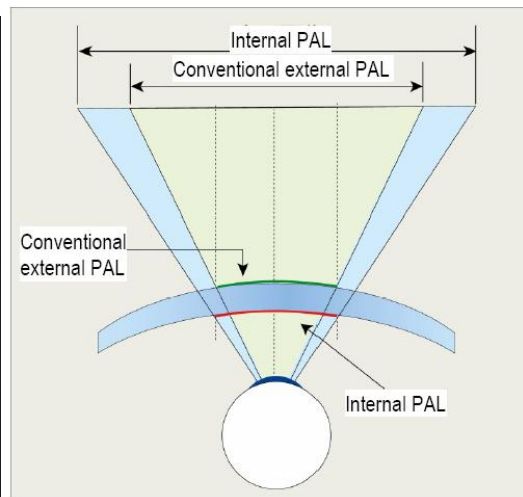
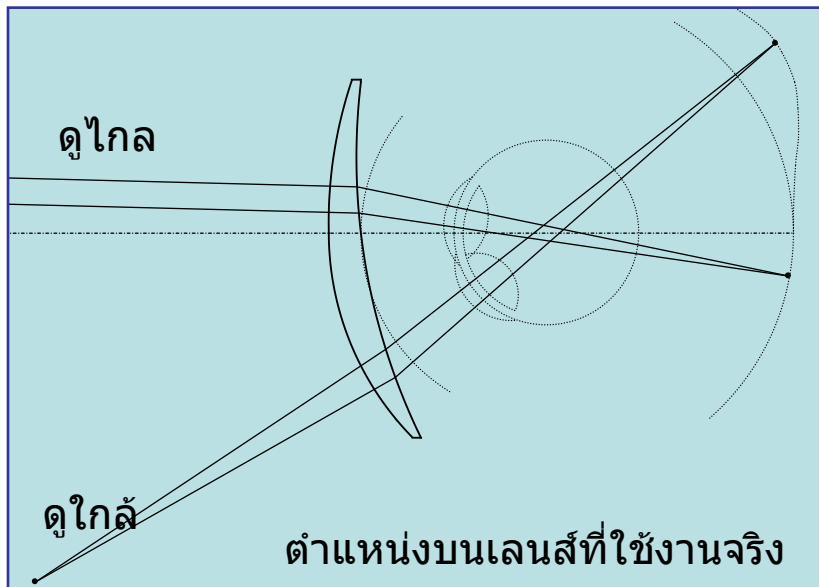
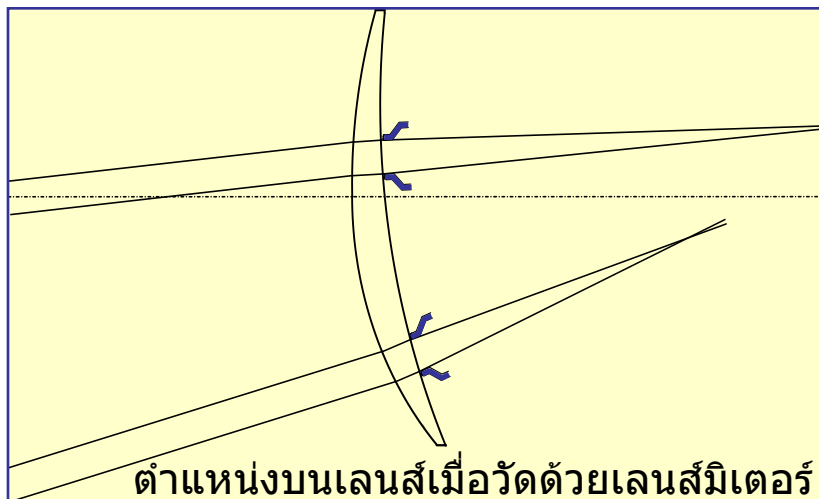
FREEDOM CLASSIC

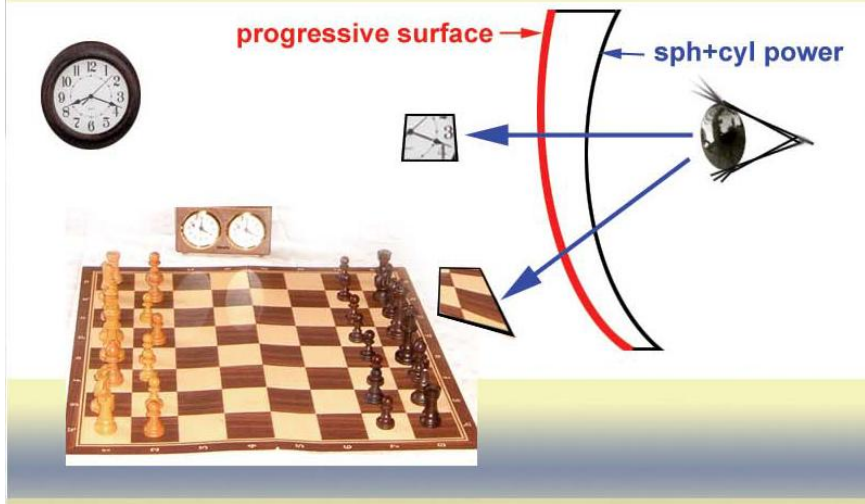CONVENTIONAL PAL front side progressive surface

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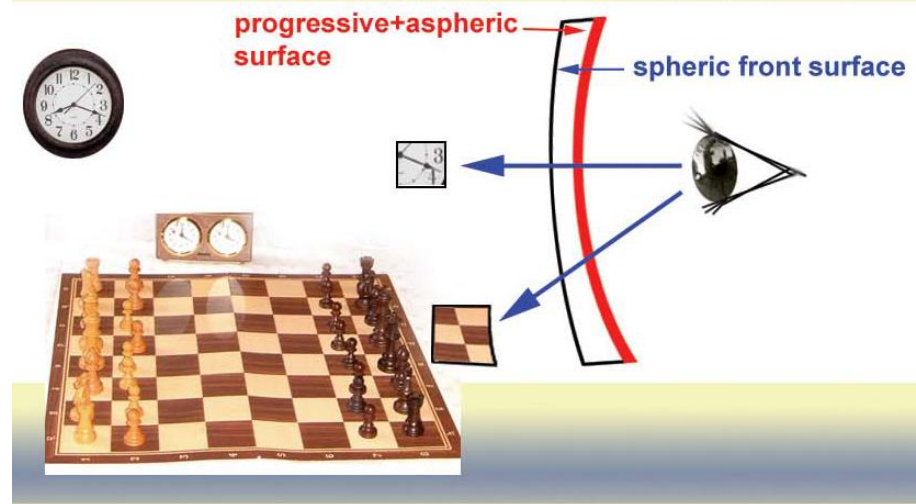
FREEDOM LENS back side progressive surface

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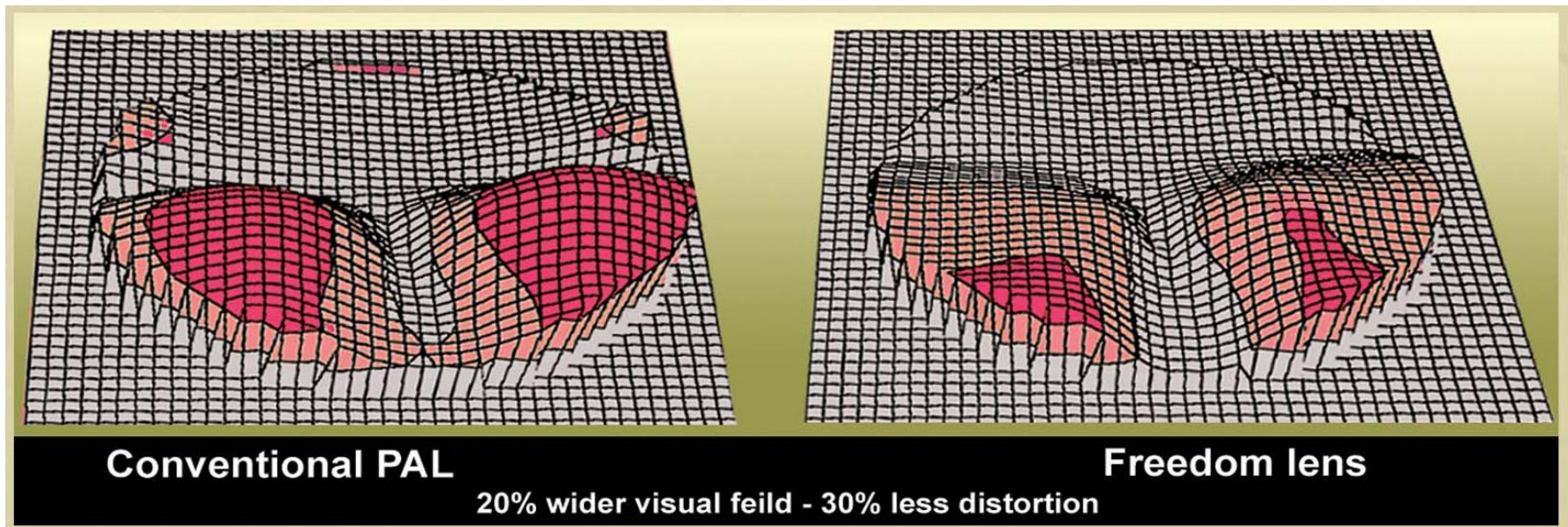HILUX OPTICAL

FREEDOM CLASSIC

CONVENTIONAL PAL : INTERMEDIATE ZONE

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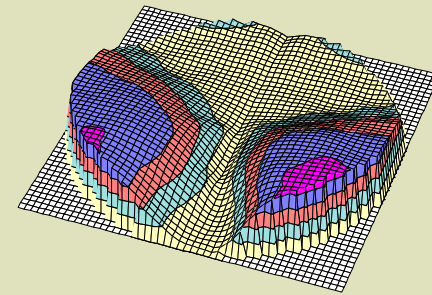
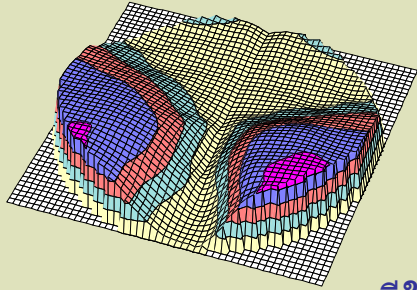
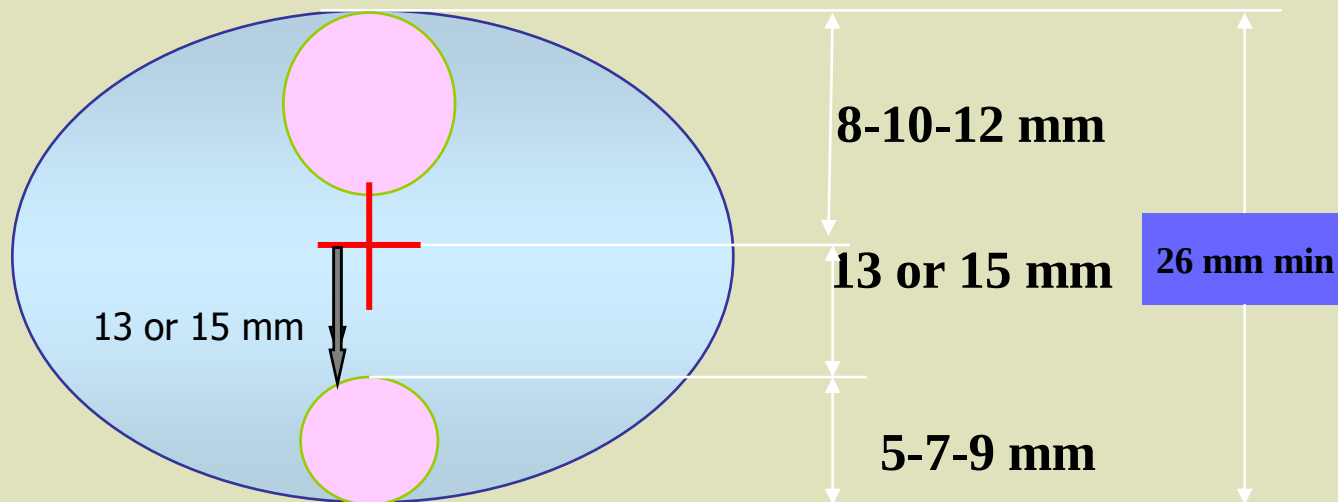
HILUX OPTICAL

Concepts :

1. Freeform Progressive Back Surface : up graded FREEDOM
2. more WAVE FRONT TECHNOLOGY - High Definition
Higher Order Aberrations eliminated : computer software ระดับสูง
3. Individualized concepts
 - power optimized : SPH-CYL-AXIS-PRISM
 - Life Style : indoor – out door – balance
 - Corridor length : 3 steps 13 – 15 – 17 mm

OUT DOOR : LFV design

BALANCE : BV design

INDOOR : LNV design

FREEDOM

Neo-Vista[®]

HILUX OPTICAL

WAVE FRONT TECHNOLOGY - High Definition

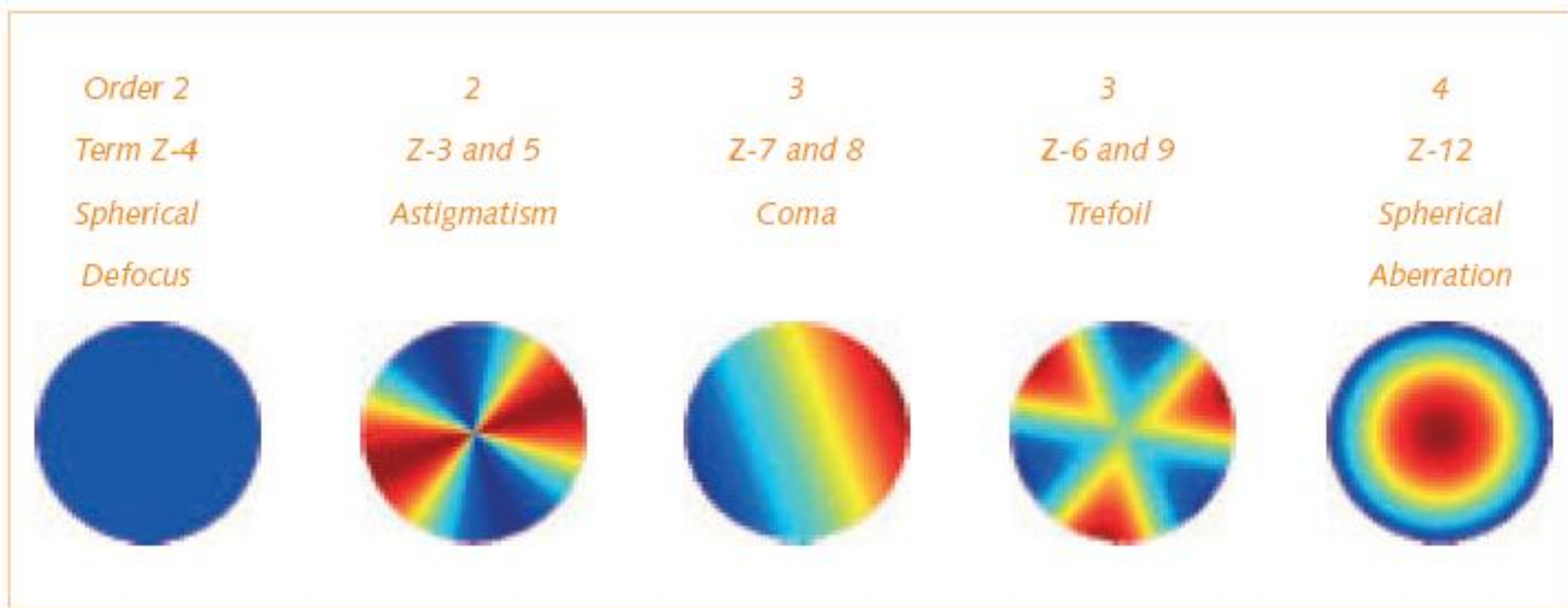
Fig. 8. Most important aberrations of the eye expressed in terms of primary refractive error modes.

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Neo-Vista

HILUX OPTICAL

WAVE FRONT TECHNOLOGY - High Definition

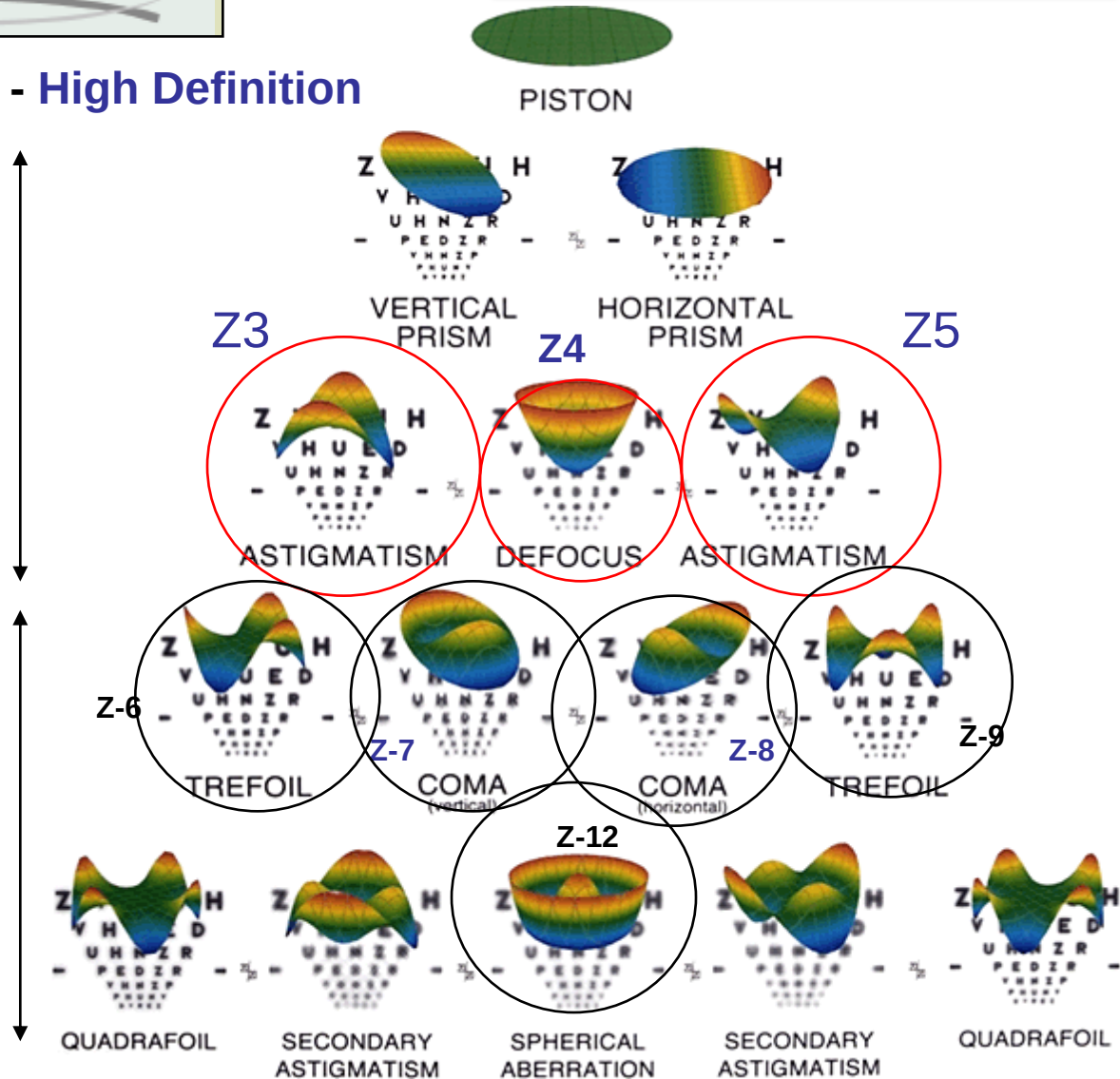
1st – 2nd : LOA

Low Order Aberrations
85% of all aberrations

Z = Zernike polynomials

3rd – 4th : HOA

Higher Order Aberrations
Almost 15% of all aberrations


FREEDOM

Neo-Vista[®]

HILUX OPTICAL

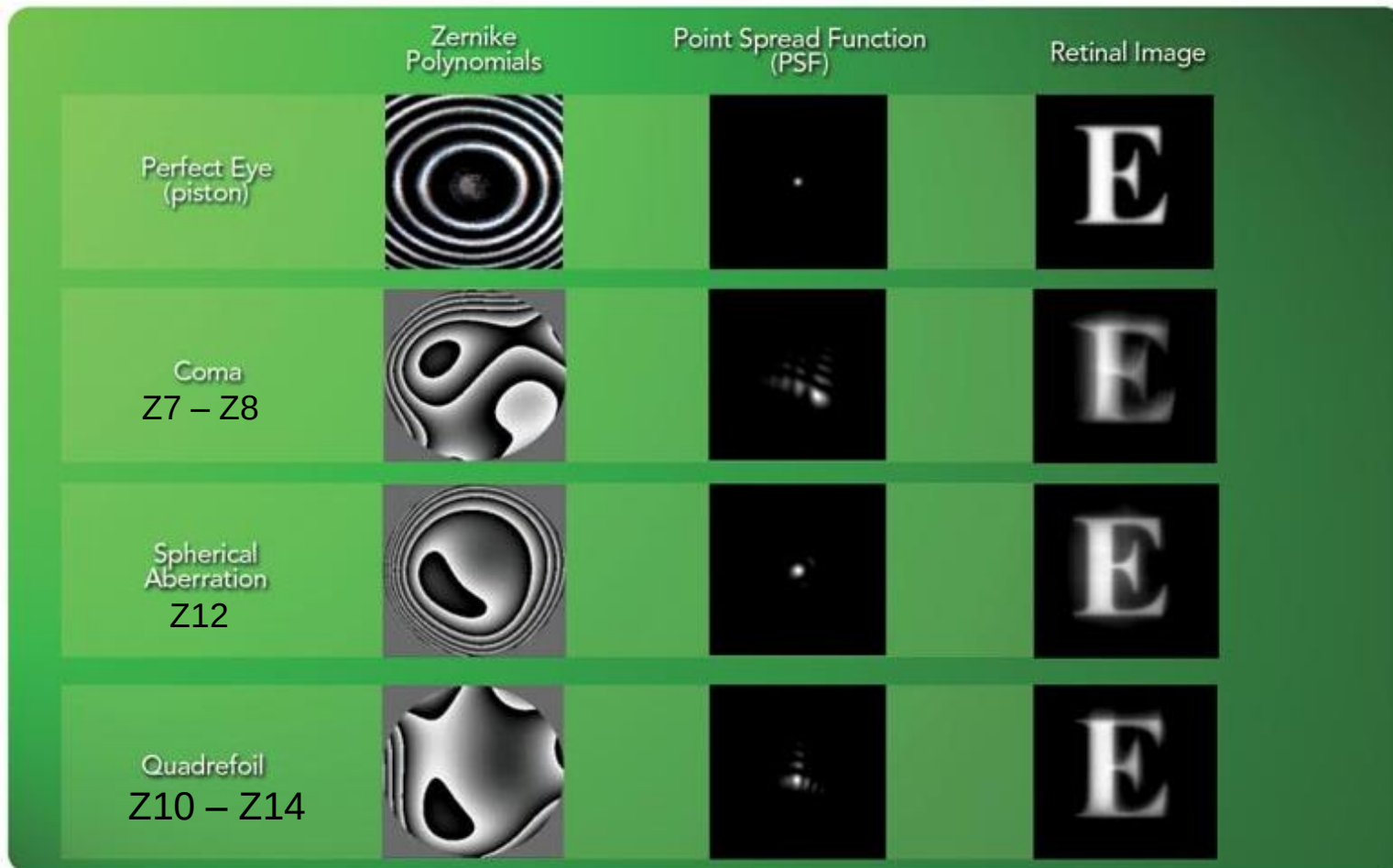
WAVE FRONT TECHNOLOGY - High Definition


FREEDOM

Neo-Vista[®]

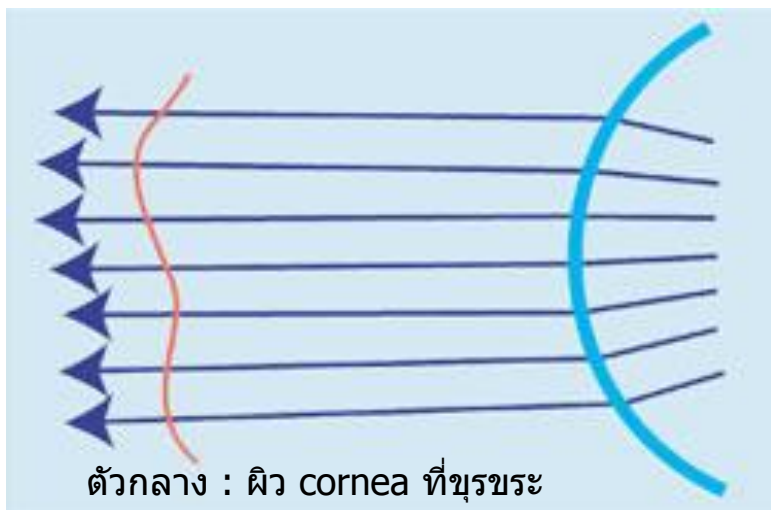WAVE FRONT TECHNOLOGY - High Definition

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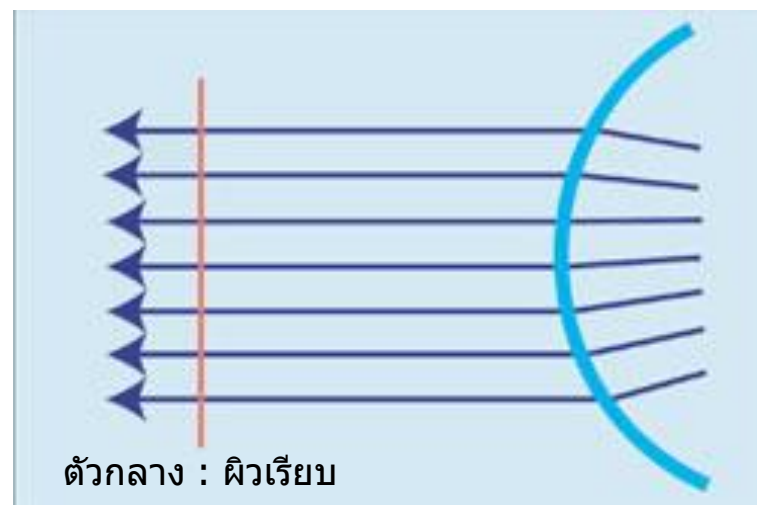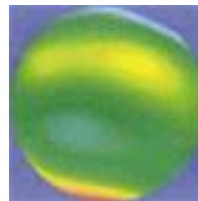
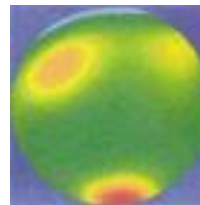
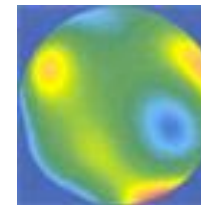
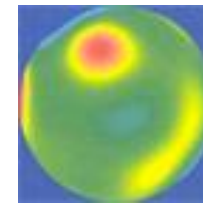
0.04 D

0.05 D

Degree of aberrations: **High** Moderate Low

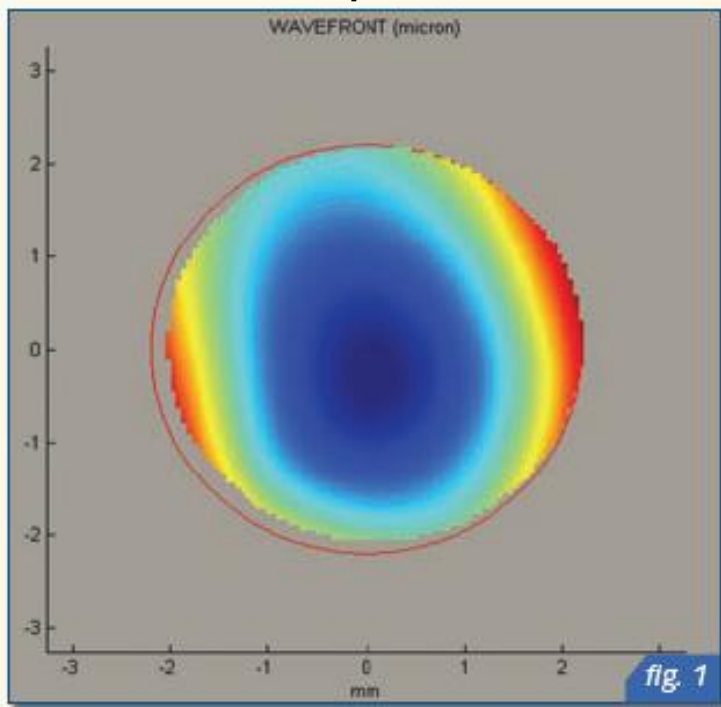
HILUX OPTICAL

FREEDOM


Neo-Vista[®]

WAVE FRONT TECHNOLOGY - High Definition


color code map : Wave Front map

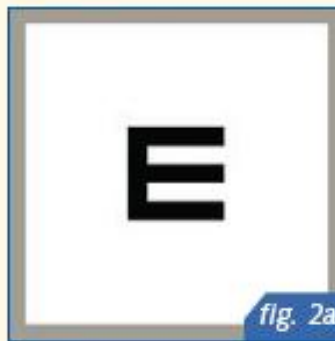Colour coded map of the optical aberrations of an eye with severe keratoconus.

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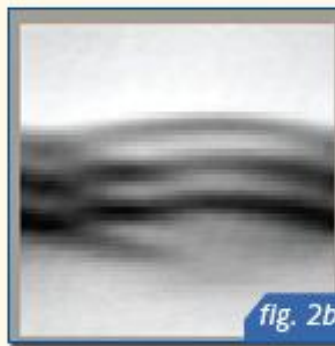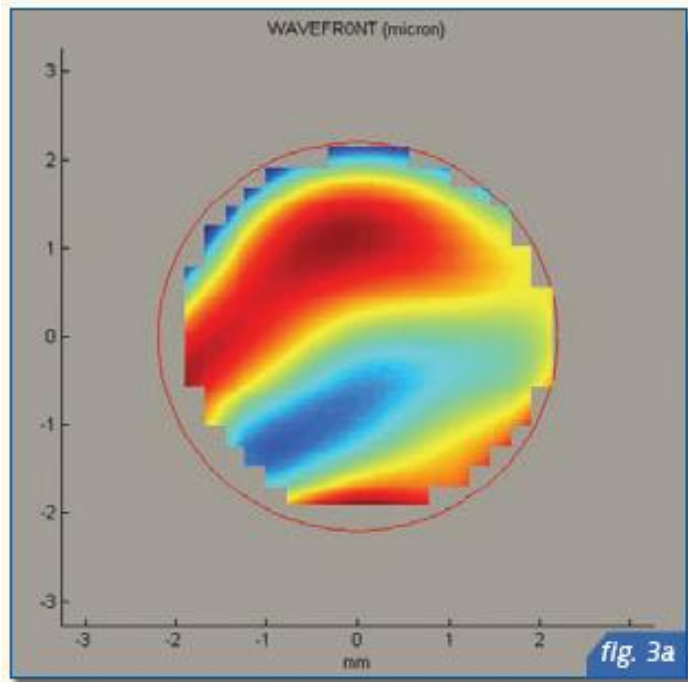
FREEDOM

Neo-Vista[®]

WAVE FRONT TECHNOLOGY - High Definition

color code map : Wave Front map

The resulting wave front with optimal spectacle correction for the keratoconus patient as presented in Fig. 1.

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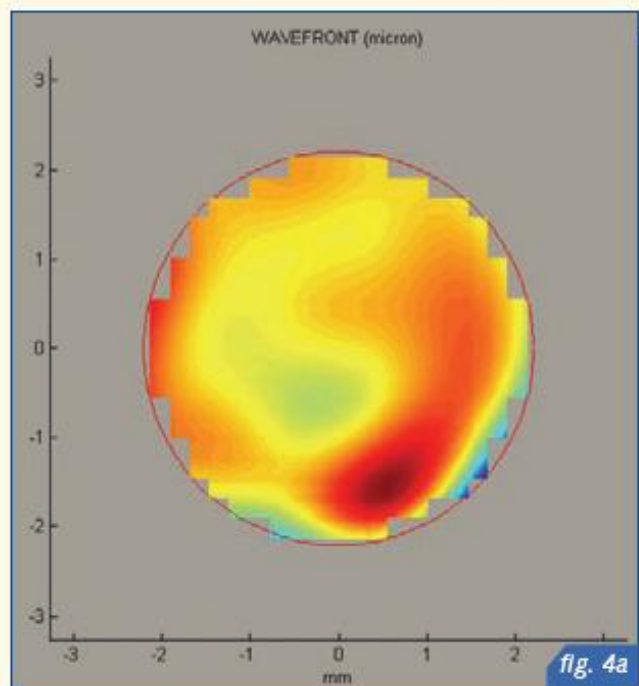
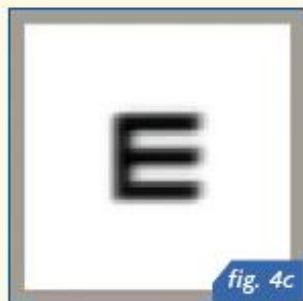
The resulting wave front with the optimal present rigid contact lens correction of the patient suffering from severe keratoconus as presented in Fig 1.

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FREEDOM

Neo-Vista®

Individualized concept : Life Style : outdoor : target group

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FREEDOM

Neo-Vista

Individualized concept : Life Style :
Outdoor : design – LFV-13

HILUX OPTICAL

FREEDOM

Neo-Vista

Individualized concept : Life Style :
Outdoor : design – LFV-15

HILUX OPTICAL

FREEDOM

Neo-Vista

Individualized concept : Life Style :
Outdoor : design – LFV-17

HILUX OPTICAL

FREEDOM

Neo-Vista

Individualized concept : Life Style : Balanced Vision design : target group

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FREEDOM

Neo-Vista

Individualized concept : Life Style :
Balance Vision : design – Balance -15

HILUX OPTICAL

FREEDOM

Neo-Vista

Individualized concept : Life Style :
Balance Vision : design – Balance -17

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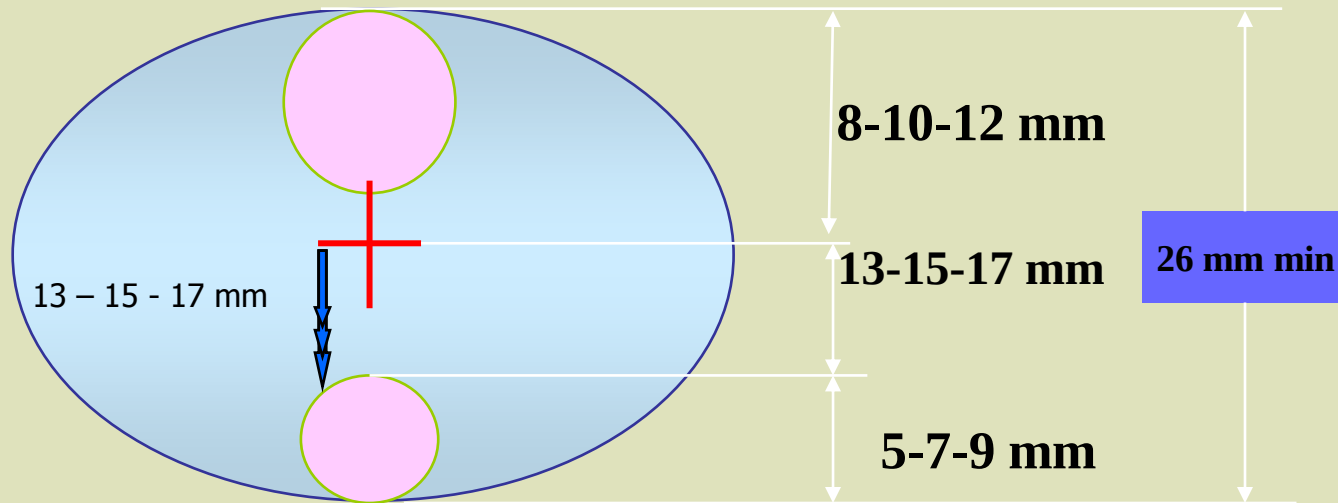
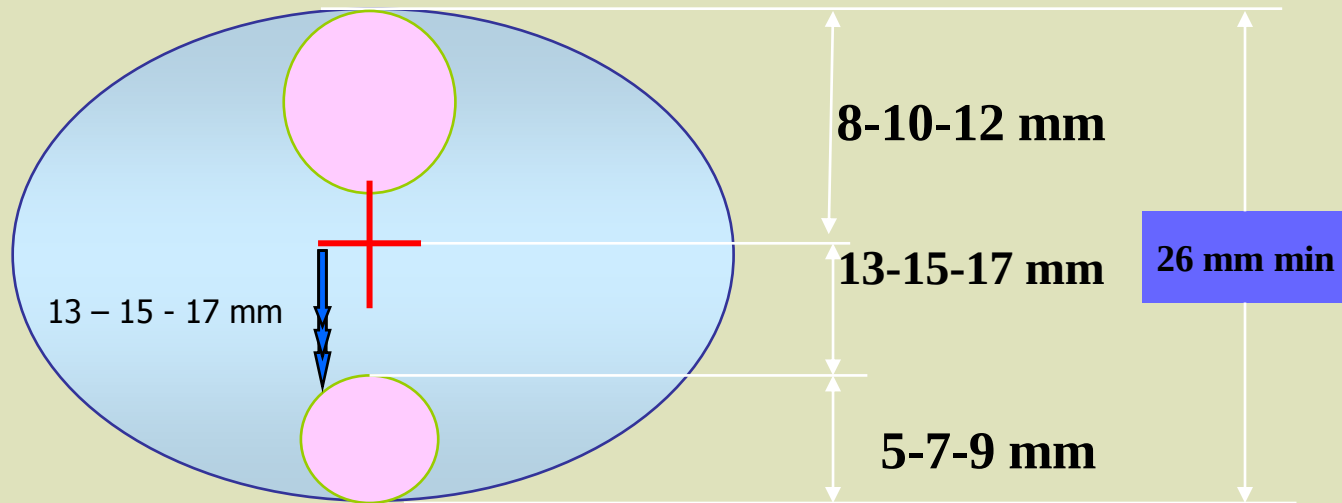
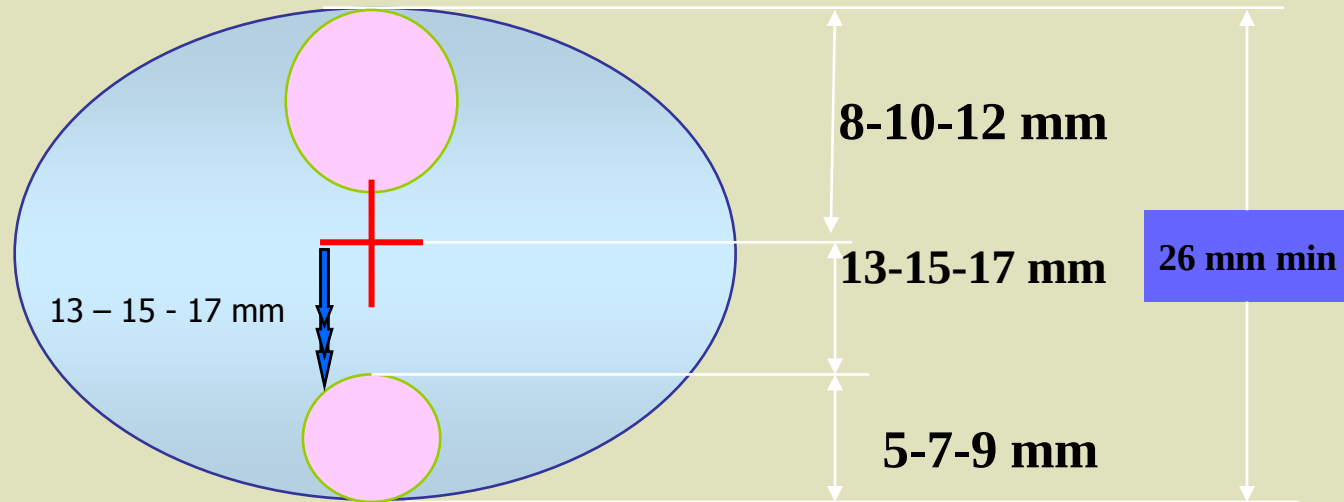
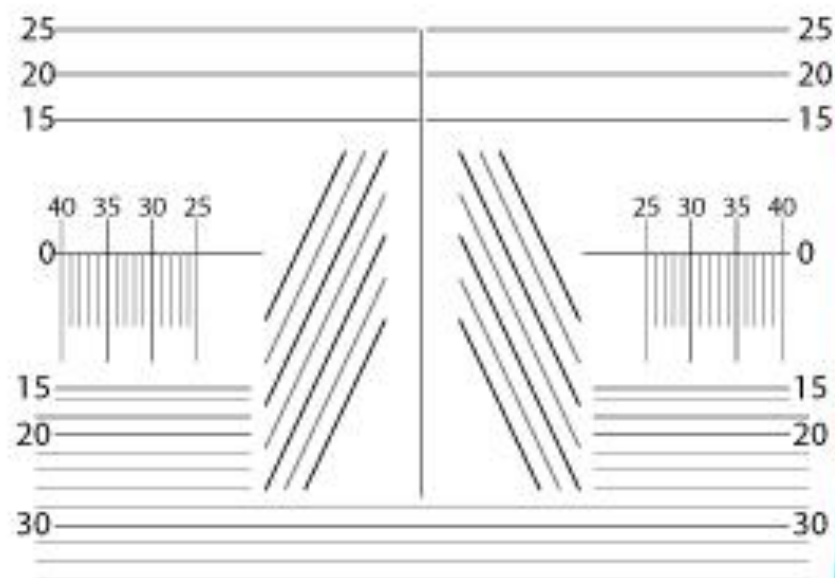
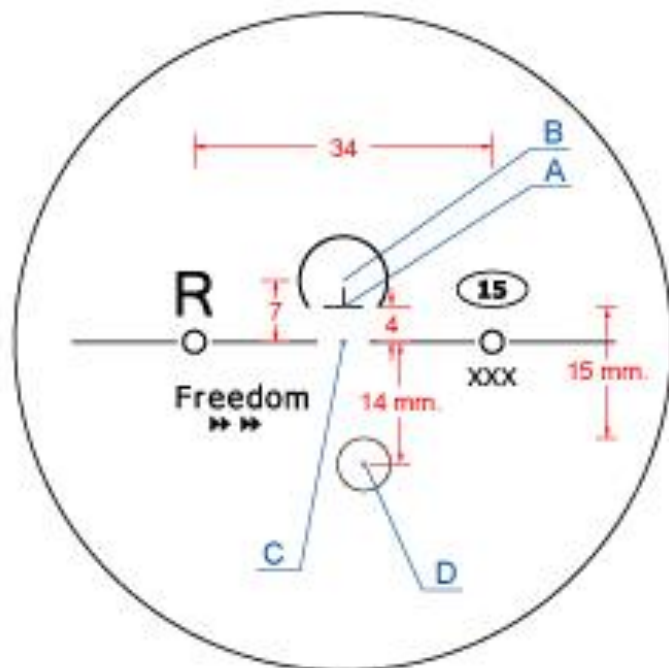
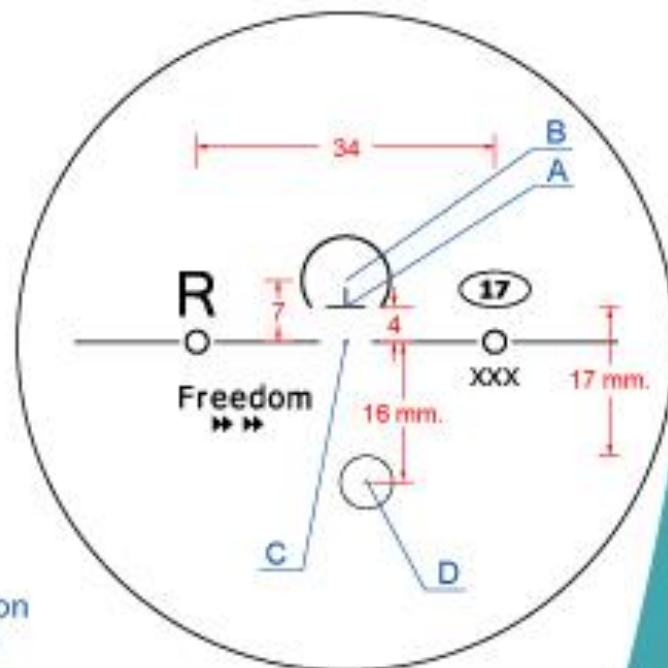
Transitions is a registered trademark of Transitions Optical, Inc.

Corridor 17

Addition
Power
Code

10=1.00
12=1.25
15=1.50
17=1.75
20=2.00
22=2.25
25=2.50
27=2.75
30=3.00
32=3.25
35=3.50

HILUX OPTICAL

A : Fitting Cross (FC)

B : Distance Reference Point (DRP)

C : Prism Reference Point (PRP)

D : Near Vision Reference Point (NRP)